

Release Notes

RescueNet Code Review, ZOLL Data Relay, ZOLL Data Retriever, and Software Development Kit ,Version 5.13

Document updated: February 3, 2010

© 2000 - 2010 by ZOLL Data Systems. All rights reserved. ZOLL Data Systems is a subsidiary of ZOLL Medical Corporation.

ZOLL, M Series, and E Series are registered trademarks of ZOLL Medical Corporation. RescueNet is a trademark of ZOLL Medical Corporation.

Bluetooth® is a registered trademark of Bluetooth SIG, Inc.

Other product and company names may be the trademarks of their respective owners.

Code Review

New Features

- Two new prompts added for AED Plus to support French changes.
- Some enhancements have been added to the parser to compensate, when possible, for bad data found on some cards. The parser has been improved in an effort to display all good cases found on a card rather than halting at the first error encountered.

Bug Fixes

- When viewing data on the CPR Quality Calculations tab, if a blank new case is opened, the tab is refreshed and no longer shows the data from the previous case.
- The logic for displaying the implanted pacemaker spike on the ECG strip was modified to not be shown when there is interference known as “noisy ECG”.
- Corrected a time sync issue that occurred when the data card was inserted hours after the case start time.
- Changed the time on 12-lead 4x3 ECG image on the 12-lead tab to the recorded time to match the defibrillator printout. This time was also updated on the report.

- Improved logic in the install which checks for Microsoft prerequisites.

ZOLL Data Retriever

New Features

- Additional dialogs have been added to alert the user that the erase card process is about to begin and that the erase process completed successfully.
- Some enhancements have been added to the parser to compensate, when possible, for bad data found on some cards. The parser has been improved in an effort to display all good cases found on a card rather than halting at the first error encountered.

Bug Fixes

- The “Object reference is not set to an instance of an object” error that occasionally occurred during upload will now be handled correctly and create cases if good cases are found on the card.

ZOLL Data Relay

Bug Fixes

- Run Number will now be displayed in the report header rather than the Patient ID.
- When the application is closed abnormally and then re-launched, any 12-lead that was previously listed as “delivered” will no longer be in the list.

Software Development Kit

Bug Fixes

- Run Number will now be displayed in the report header rather than the Patient ID.
- Changed the time on 12-lead 4x3 ECG PNG image to be the recorded time to match the defibrillator printout.